

Wrist Brace

INDICATIONS FOR USE

- » Over use injuries
- » Healing fractures
- » Ligament strains
- » Rheumatoid arthritis/osteoarthritis

MEASUREMENT

- » Wrist circumference at narrowest point

FITTING INSTRUCTIONS

- » Undo all contact closures
- » Remove aluminium palmar stay and bend to required shape
- » Reinsert palmar stay and cover with end tab
- » Place support on underside of wrist and align palmar stay to palm of hand
- » Fasten wrist closures to ensure snug fit

WASHING INSTRUCTIONS

- » Remove metal stays and hand wash and air dry
- » Air dry

Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty.



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR
E: info@beagleortho.com W: beagleortho.com T: 01254 268 788