

# Wrist Thumb Brace

## INDICATIONS FOR USE

- » Wrist/thumb trauma
- » Wrist/thumb over use injury
- » Stabilising hyper- or hypo-tonic wrist/thumb flexors

## MEASUREMENT

- » Wrist circumference at narrowest point

## FITTING INSTRUCTIONS

- » Undo all contact closure
- » Remove palmar stay and thumb stay
- » Adjust palmar stay as required
- » With distal tip of thumb stay in line with the distal interphalangeal joint and thumb in neutral, shape stay as required
- » Replace palmar and thumb stays
- » Place support on underside of wrist and align palmar stay to palm of hand
- » Fasten wrist closure to ensure snug fit
- » Finally wrap thumb round thumb and stay to provide thumb support as required

## WASHING INSTRUCTIONS

With the palmar and thumb stays removed:

- » Neoprene wrist/thumb braces may be machine washed at 40°C and air dried
- » Elastic wrist/thumb braces should be hand washed and air dried

Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty.



**Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR  
E: [info@beagleortho.com](mailto:info@beagleortho.com) W: [beagleortho.com](http://beagleortho.com) T: 01254 268 788