

NEURODYN®-COMFORT

 **SPORLASTIC**

Fußheberorthese mit Einhandverschluss

Dorsal flexor orthesis with single-hand closure

Orthèse des releveurs du pied avec fermeture
d'une seule main


Vendaje funcional con cierre sencillo

Ortesi peroneale a chiusura con una mano sola

Voetorthese met eenhands sluiting







Orteza prostowników stopy z wygodnym zapięciem

Fotlyftsortos med enhandsstängning

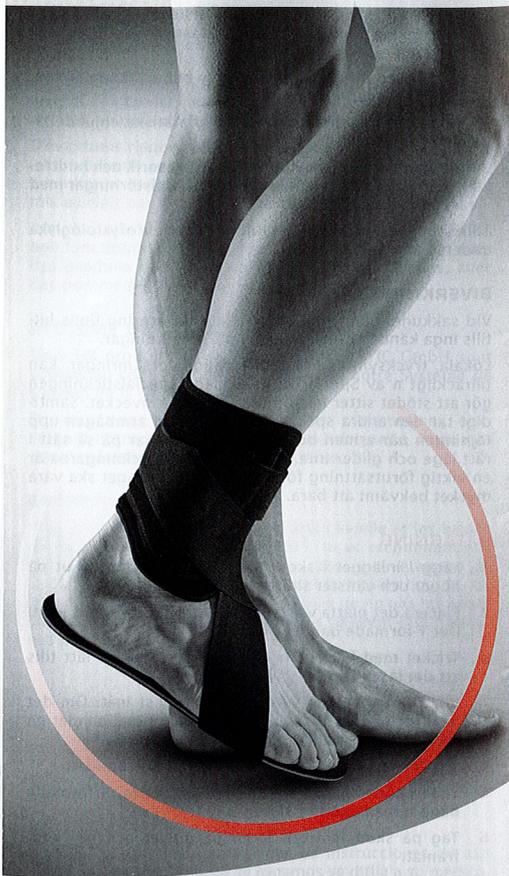

NEURODYN®-COMFORT
SPORLASTIC®

Fußheberorthese mit Einhandverschluss
 Dorsal flexor orthesis with single-hand closure
 Orthèse des releveurs du pied avec fermeture
 d'une seule main
 Vendaje funcional con cierre sencillo
 Ortesi peroneale a chiusura con una mano sola
 Voetorthese met eenhands sluiting
 Orteza prostowników stopy z wygodnym zapięciem
 Fotlyftsortos med enhandsstängning

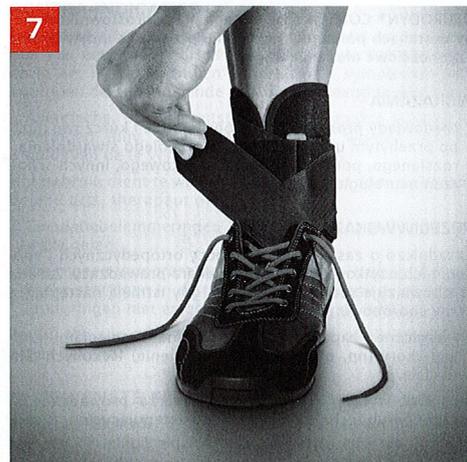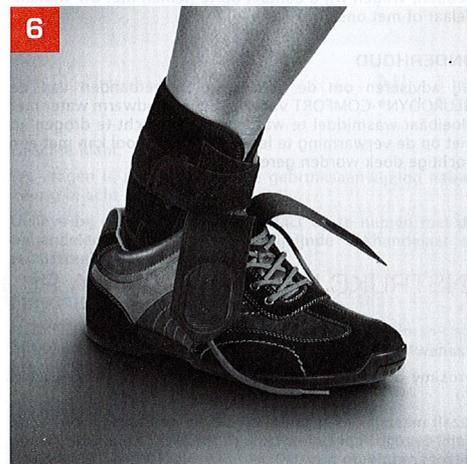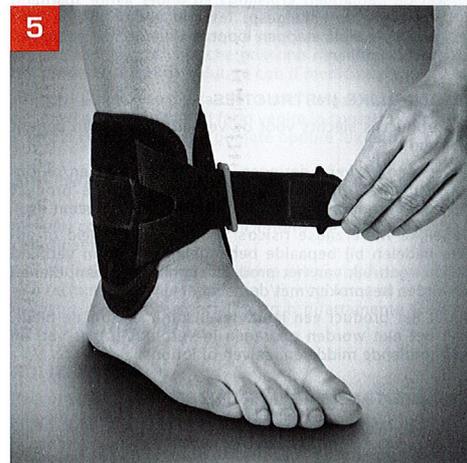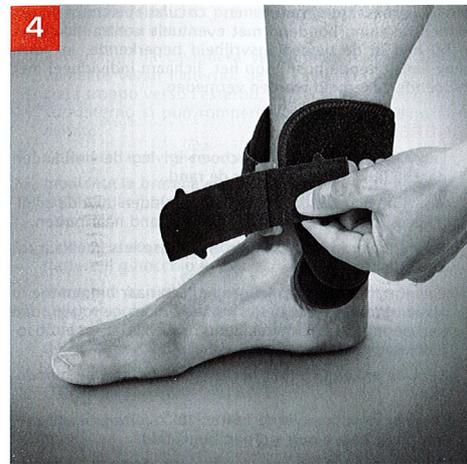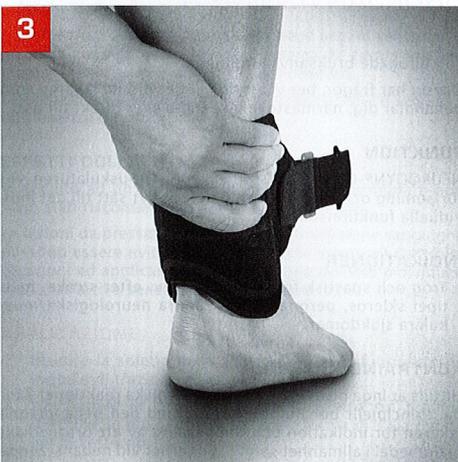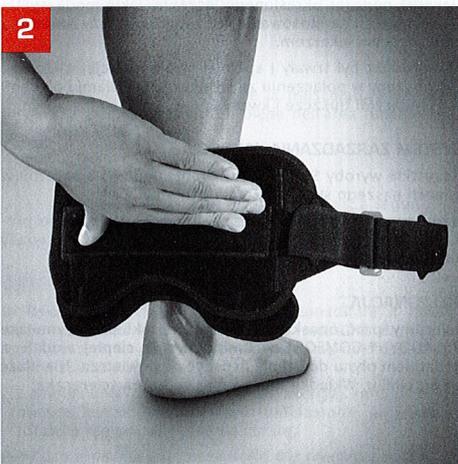















SPORLASTIC GmbH
 Neurodyn-Produkte
 Weststraße 1, 72525 Marbach, Germany
 Telefon: +49 7141 9330-0
 E-Mail: info@sporlastic.de, www.sporlastic.de

INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

NEURODYN®-COMFORT supports the dorsal flexor muscles affected by paralysis and actively helps the foot return to its individual functional area.

INDICATIONS

- Paralysed and spastic dorsal flexor paresis (e.g. after a stroke, multiple sclerosis, peroneus paresis and other neurological/muscular diseases).

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.

Utilisation requires a prior check-up by your doctor, therapist or technician for:

- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- 1 – Insert the insole into your shoe so that the lifting straps protrude left and right.
- 2 – Position the flat ankle clasp centrally over the Achilles tendon so that the Y-shaped strap faces outwards.
- 3 – The trick with the click: Press the ankle clasp gently together until it clicks shut.
Important: Only press the ankle clasp together inwards. If you overbend it outwards, the metal strip sewn into it may break. This must be avoided.
- 4 – Close the ankle clasp with the inside Velcro strap.
- 5 – The ankle clasp can be readjusted using the outside, Y-shaped strap with the deflection ring.
- 6 – Put on your shoe, place your foot on the floor and bend your knee forwards.
- 7 – Close the lifting straps crosswise and under slight tension onto the ankle clasp. Take several steps to test how your forefoot is raised. Readjust the strap if required.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it

must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that you wash the ankle clasp and lifting straps on the NEURODYN®-COMFORT gently in lukewarm water using liquid detergent and air-dry them. Do not place them on radiators. The insole can be cleaned using a damp cloth.