

Product Codes

Sizing Information

Size	Upper arm Circ (mm)	Code
Small	280 - 330	0583
Medium	330 - 380	0585
Large	380 - 430	0587
XLarge	430 - 480	0588



DOUBLE SHOULDER BRACE FITTING INSTRUCTIONS

Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty.



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR
E: info@beagleortho.com W: beagleortho.com T: 01254 268 788

INDICATIONS FOR USE

- » Shoulder Instability
- » Lax Capsule
- » Rhomboid and Trapezius Strains



1 Place arm into cuff



2 Secure the hook & loop fastening



3 Take across back of neck



4 Place arm into secon cuff and secure the hook & loop fastening



5 To achieve more abduction undo the Velcro on nape of neck, tighen and re-secure