

## Upper Limb

Bea Shoulder Sling			
0663	0665	0667	0668

### Description

An orthosis which provides immobilisation following injury or surgery with an integrated neck pad that provides additional patient comfort. The sling features an adjustable neck and waist strap. Elbow mobilisation is possible without complete sling removal and the thumb loop effectively maintains sling placement.

### Intended Purpose

To provide upper limb support for an array of injuries and conditions, including Pre or Postoperative treatment for: CVA, Post-trauma, Postoperative, Immobilisation following injury or surgery.

### Sizing

Measure the length from the elbow point to the start of the little finger

### ⚠ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Place the injured arm into the sling, ensuring the elbow and wrist are supported securely to avoid discomfort **See Fig.1.**
2. Run the shoulder strap up from the injured elbow, round the neck and over the uninjured shoulder. Feed it through the D-ring on the top of the sling and secure with the hook and loop fastening **See Fig.2.**
3. Attach the immobilisation strap on to the side of the sling **See Fig.3.**
4. **To use as a Shoulder Immobiliser:** Feed the immobilisation strap through the D-ring on the elbow and secure with the hook and loop fastening **See Fig.3.**
5. Run the immobilisation strap around the waist and feed it through the D-ring on the front of the sling. Secure with the hook and loop fastening on the waist. **See Fig.4.**
6. Place the thumb into the soft thumb loop to maintain optimal positioning and support the hand **See Fig.5.**



### **Contraindications**

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

### **Warnings and Precautions**

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

### **Washing and care instructions**

- Cover all the hook and loop straps
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

### **Product composition**

Cotton, Hook and Loop, Plastic Buckle

### **Storage and transport conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

### **Recycling and disposal**

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

### **Serious incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

### **Wash symbols**



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




**Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

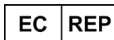

**International Associates Auditing & Certification Limited**

The Black Church, St. Mary's Place, Dublin, D07 P4AX