

Arthritic Knee Sleeve

INDICATIONS FOR USE

- » Arthritis

MEASUREMENT

- » Circumference of knee

FITTING INSTRUCTIONS

- » Place the foot through the brace, with the widest part at the top and the padded section on the front
- » Pull the brace up to cover the knee with equal amounts of the brace above and below the knee

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

Size	Knee Circ	Code
Small	280 - 320	803/AKS
Medium	320 - 390	805/AKS
Large	380 - 440	807/AKS
XLarge	440 - 490	808/AKS
XXLarge	490 - 540	809/AKS



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: info@beagleortho.com W: beagleortho.com T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.