

Bea Mat Belt

INDICATIONS FOR USE

- » Sacroiliac support throughout all stages of pregnancy
- » Pubic support
- » Backache and Fatigue

FITTING INSTRUCTIONS

- » Open the hook and loop fastening
- » Wrap the brace around the lower back, with the flexible steels lying on either side of the spine
- » Wrap the soft elastic underneath the belly alleviating the weight on the lower spine and lifting the belly and close the hook and loop fastening

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

Hip Circ (mm)	6" Bea Mat Belt	8" Bea Mat Belt
900 - 960	DN/900	BM/900
960 - 1020	DN/960	BM/960
1020 - 1080	DN/1020	BM/1020
1080 - 1160	DN/1080	BM/1080
1160 - 1260	DN/1160	BM/1160
1260 - 1360	DN/1260	BM/1260
1360 - 1460	DN/1360	BM/1360



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR
E: info@beagleortho.com W: beagleortho.com T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.