

Clavicle Splint

INDICATIONS FOR USE

- » Clavicle Fracture
- » Clavicle Injury
- » Shoulder Retraction
- » General Posture Support

MEASUREMENT

- » Measure from centre of shoulder blades, over shoulder, under axilla (armpit) and back to centre of shoulder blades

FITTING INSTRUCTIONS

- » Unclip both straps
- » Position the square pad, on the back, between the shoulder blades.
- » Run the straps over the shoulders and under the arms and attach to the non-slip clips. The straps can be adjusted as required.

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

| Size | Shoulder - Axilla (mm) | Codes |
|-------------|-------------------------------|--------------|
| Small | 560 - 640 | CLA/03 |
| Medium | 740 - 720 | CLA/05 |
| Large | 720 - 800 | CLA/07 |
| XLarge | 800 - 880 | CLA/08 |



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: info@beagleortho.com W: beagleortho.com T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.