

# Paediatric D-Ring Wrist Brace

## INDICATIONS FOR USE


- » Green Stick Fracture
- » Buckle Fracture
- » Healing Wrist Fractures & Ligament Injuries
- » Post Injury

## MEASUREMENT

- » Wrist circumference at narrowest point

## FITTING INSTRUCTIONS

- » Undo all hook & loop straps
- » If required you can bend the aluminium palmar bar to individual hand shape. To do this remove bar from brace place bar along the underside of wrist into the palm. If there are spaces bend as required
- » Place the hand into the brace ensuring the stockinette is above the hand, as per the image.
- » Secure hook & loop fastening to provide a snug fit

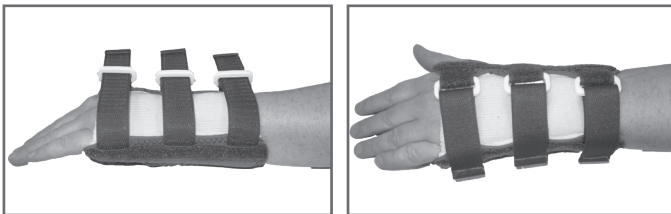

## WASHING INSTRUCTIONS

- » Remove metal stays
- » Hand wash
- » Air dry



## Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: [info@beagleortho.com](mailto:info@beagleortho.com) W: [beagleortho.com](http://beagleortho.com) T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd  
Sutton House, Bishopstown Road  
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.