

# Soft Collar

## INDICATIONS FOR USE

- » Neck strain/sprain
- » Arthritis
- » Whiplash injuries




## MEASUREMENT

- » Circumference of neck plus 4 inches
- » Depth from under ear to the collar bone

## FITTING INSTRUCTIONS

Once correct size has been chosen:

- » Offer collar up to the neck, centre the collar with chin secured in the shaped recess
- » Secure at the back with the hook and loop fastening
- » Adjust as necessary to secure the cervical/neck region

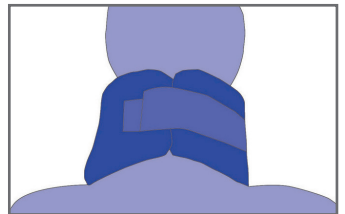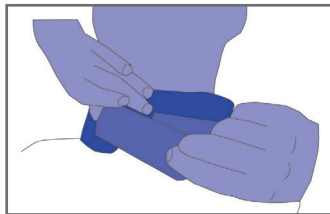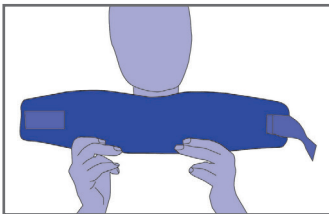

## WASHING INSTRUCTIONS

- » Hand wash
- » Air dry


## Sizing Information

Size	Code - White	Code - Black
17" X 2.5"	SFC/171	SFC/171.BLK
17" X 3"	SFC/172	SFC/172.BLK
17" X 3.5"	SFC/173	SFC/173.BLK
18" X 2.5"	SFC/181	SFC/181.BLK
18" X 3"	SFC/182	SFC/182.BLK
18" X 3.5"	SFC/183	SFC/183.BLK
19" X 2.5"	SFC/191	SFC/191.BLK
19" X 3"	SFC/192	SFC/192.BLK
19" X 3.5"	SFC/193	SFC/193.BLK
20" X 2.5"	SFC/201	SFC/201.BLK
20" X 3"	SFC/202	SFC/202.BLK
20" X 3.5"	SFC/203	SFC/203.BLK
22" X 2.5"	SFC/221	SFC/221.BLK
22" X 3"	SFC/222	SFC/222.BLK
22" X 3.5"	SFC/223	SFC/223.BLK
24" X 2.5"	-	SFC/241.BLK
24" X 3"	-	SFC/242.BLK
24 X 3.5"	-	SFC/243.BLK



### Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR  
 E: info@beagleortho.com W: beagleortho.com T: 01254 268 788 F:01254 268789

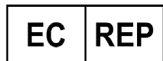

European Healthcare & Device Solutions Ltd  
 Sutton House, Bishopstown Road  
 Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.