

## Upper Limb

### Clavicle Splint

CLA/03	CLA/05	CLA/07	CLA/08
--------	--------	--------	--------

#### Description

An orthosis designed to provide optimal support and immobilisation of the collarbone while recovering from a broken or injured clavicle. The brace helps maintain ideal posture by pulling evenly over the scapula's. The slimline design means the brace is unnoticeable when worn under clothing and the slip-resistant buckles ensure patient compliance. The brace also features soft foam padding to ensure no underarm rash occurs, improving comfort and compliance.

#### Intended Purpose

To provide upper limb support for an array of injuries and conditions, including Pre or Postoperative treatment for: Clavicle Fracture, Clavicle Injury, Shoulder retraction, General posture support.

#### Sizing

Measure from the centre of the shoulder blades, over the shoulder, under the axilla and back to the centre of the shoulder blades.

#### ⚠ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Unclip both straps and position the square pad, on the back, between the shoulder blades. **See Fig.1.**
2. Run the strap over the shoulder and under the arm and attach the buckle to the non-slip clip. **See Fig.2.**
3. Repeat on the opposite side. **See Fig.3.**
4. The straps can be adjusted as required. **See Fig.4.**



### **Contraindications**

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

### **Warnings and Precautions**

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

### **Washing and care instructions**

- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use

### **Product composition**

Velfoam Laminate, Hook and Loop, Webbing, Aluminium

### **Storage and transport conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

### **Recycling and disposal**

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

### **Serious incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

### **Wash symbols**



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




**Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

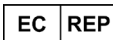

**International Associates Auditing & Certification Limited**

The Black Church, St. Mary's Place, Dublin, D07 P4AX