

# Spinal

<b>Bea Comfort TLSO</b>		
BCOM/02	BCOM/03	BCOM/05
BCOM/07	BCOM/08	BCOM/09

## Description

An orthosis designed specifically for patients who will not conform to a traditional hard shell brace. Our fabric TLSO is constructed from a white ventilated elastic material for all day breathability. The ventilated material optimises air flow whilst providing gentle compression. The brace also features Velfoam pads and rigid steels.

## Intended Purpose

To provide back support for an array of injuries and conditions, including Pre or Postoperative treatment for: Spondylolisthesis, Spondylosis, Lower back and general back pain.

## Sizing

Measure the circumference of the waist.

## △ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Undo the hook and loop fastenings on the front of the brace and on the long shoulder straps **See Fig.1.**
2. Align the brace around the lower back with the padded section lying in the middle over the spine **See Fig.2.**
3. Wrap the brace over in front and fasten with the hook and loop fastenings **See Fig.3.**
4. Bring the long straps over the shoulders and ensure the padding sits on the bony part at the front of the shoulder **See Fig.4.**
5. Pass the straps under the arms and cross them over at the back **See Fig.5.**
6. Bring the straps back around to the front and fasten them to each other using the hook and loop fastenings **See Fig.6.**



### **Contraindications**

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

### **Warnings and Precautions**

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

### **Washing and care instructions**

- Cover all the hook and loop straps and remove metal bars
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

### **Product composition**

Velfoam, Elastic, Hook and Loop, Latex

### **Storage and transport conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

### **Recycling and disposal**

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

### **Serious incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

### **Wash symbols**



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron



Contains natural rubber latex




**Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

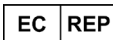

**International Associates Auditing & Certification Limited**

The Black Church, St. Mary's Place, Dublin, D07 P4AX