

## Spinal

Bea Mat Belt						
DN/900	DN/960	DN/1020	DN/1080	DN/1160	DN/1260	DN/1360
BM/900	BM/960	BM/1020	BM/1080	BM/1160	BM/1260	BM/1360

### Description

An orthosis to support throughout pregnancy. The Bea Mat Belt is designed to relieve backache and fatigue throughout all stages of pregnancy. The belt is constructed from a durable leno weave elastic, providing superior compression. The brace features a wide elastic that passes underneath the stomach, alleviating the weight on the lower spine and lifting the belly. The touch and close hook and loop fastenings adjust as the bump grows. Available in two depths – 6" and 8". The 8" belt features a small Velfoam Lumbar pad for additional comfort.

### Intended Purpose

To provide back and pelvic support for an array of injuries and conditions, including Pre or Postoperative treatment for: Pubic Support, Backache and fatigue, Sacroiliac support throughout all stages of pregnancy.

### Sizing

Measure the circumference of the waist.

### Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Undo the hook and loop fastening **See Fig.1.**
2. Align the belt to the midline of the lower back, with the flexible steels lying on either side of the spine **See Fig.2.**
3. Wrap the soft elastic underneath the belly alleviating the weight on the lower spine and lifting the belly **See Fig.3.**
4. Close the hook and loop fastening **See Fig.4.**



### **Contraindications**

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

### **Warnings and Precautions**

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

### **Washing and care instructions**

- Cover all the hook and loop straps and remove metal bars
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

### **Product composition**

Elastic, Hook and Loop, Latex

### **Storage and transport conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging.

### **Recycling and disposal**

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

### **Serious incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

### **Wash symbols**



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron



Contains natural rubber latex



### **Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017



### **International Associates Auditing & Certification Limited**

The Black Church, St. Mary's Place, Dublin, D07 P4AX