

Spinal

Tri Belt			
TRI/900	TRI/960	TRI/1020	TRI/1080
TRI/1160	TRI/1260	TRI/1360	TRI/1460

Description

An orthosis to support throughout pregnancy. The Tri Belt promotes correct posture and thereby allowing an active lifestyle to be continued. The belt features a three strap design, allowing the belt to be worn in a choice of configurations to achieve an optimal fit. The belt contains flexible plastic stays in the back for increased support without being too restrictive with a back pad for added comfort. The hook and loop fastenings allow for adjustment as the bump grows.

Intended Purpose

To provide back and pelvic support for an array of injuries and conditions, including Pre or Postoperative treatment for: Pubic support, Backache and fatigue, Sacroiliac support throughout all stages of pregnancy.

Sizing

Measure the circumference of the hips.

⚠ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Open the front hook and loop fastening **See Fig.1.**
2. Align the belt to the midline of the lower back, with padding lying in the middle over the spine **See Fig. 2.**
3. Secure the front strap with the hook and loop fastenings to secure the belt **See Fig.3.**
4. The three front straps are designed to be worn in a choice of configurations, over and underneath the belly, to alleviate the weight on the lower spine and lifting the belly



Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

Product composition

Elastic, Plastic, Hook and Loop, Latex

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron



Contains natural rubber latex




Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

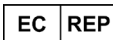

International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX