

Spinal

Soft Cervical Neck Collar				
SFC/171	SFC/172	SFC/173	SFC/181	SFC/182
SFC/183	SFC/191	SFC/192	SFC/193	SFC/201
SFC/202	SFC/203	SFC/221	SFC/222	SFC/223
SFC/171.BLK	SFC/172.BLK	SFC/173.BLK	SFC/181.BLK	SFC/182.BLK
SFC/183.BLK	SFC/191.BLK	SFC/192.BLK	SFC/193.BLK	SFC/201.BLK
SFC/202.BLK	SFC/203.BLK	SFC/221.BLK	SFC/222.BLK	SFC/223.BLK

Description

An orthosis constructed from a light density white or black velfoam with a hard composite which ensures comfort while maintaining support. Available in a range of lengths and depths to obtain the optimal fit. Adjustable fasteners maintain the desired compression to obtain much-needed pain relief and the contoured shape provides a snug fit.

Intended Purpose

To provide neck support for an array of injuries and conditions, including Pre or Postoperative treatment for: Arthritis, Neck strain/sprain, Whiplash injuries.

Sizing

For Length, measure the circumference around the neck and add 4". To size the depth, measure from under the ear to where the neck meets the body.

⚠ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Offer the collar up to the neck, centre the collar with the chin secured in the shaped recess **See Fig.1.**
2. Secure the collar at the back of the neck with the hook and loop fastening **See Fig.2.**
3. Adjust as necessary to secure the cervical/neck region **See Fig.3.**



Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps and remove metal hinges
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

Product composition

Velfoam, Stockinette, Hook and Loop

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

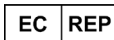

International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX