

## Knee & Lower Limb

<b>Stability Knee O, Stability Knee U</b>			
<b>SK/023</b>	<b>SK/025</b>	<b>SK/027</b>	<b>SK/028</b>
<b>SK/043</b>	<b>SK/045</b>	<b>SK/047</b>	<b>SK/048</b>

### **Description**

An orthosis with reinforced shaped buttresses centred around the patella, providing targeted compression and support while keeping the kneecap in place, preventing displacement without restricting movement. Patella Stabiliser O features a doughnut shaped buttress and Patella Stabiliser U features a horseshoe shaped buttress.

### **Intended Purpose**

To provide knee and lower limb support for an array of injuries and conditions, including Pre or Postoperative treatment for: Arthritis, Aching joints, Patella disorders that need efficient control and Weak and injured knees.

### **Sizing**

Measure the circumference of the knee at mid-patella, the thigh 152mm above mid-patella and the calf 152mm below mid patella.

### **⚠ Fitting Instructions**

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Place the foot through the brace with the widest part of the brace at the top and the hole and padded circle at the front
2. Pull the brace up to cover the knee, ensuring the open circle sits at the middle of the knee and the padded circle surrounds the knee
3. Fasten the straps through the D ring's and back over onto themselves to secure

### **Contraindications**

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

### **Warnings and Precautions**

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the brace is not over tight perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges

### **Washing and care instructions**

- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use

### **Product composition**

Neoprene, Steel Springs, Hook and Loop, Plastic

### **Storage and transport conditions**

Store in a cool, dry place out of direct sunlight, in the original packaging

### **Recycling and disposal**

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws

### **Serious incident**

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside

### **Wash symbols**



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




**Beagle Orthopaedic**

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: [info@beagleortho.com](mailto:info@beagleortho.com) t: +44 (0) 1254 268 788 w: [beagleortho.co.uk](http://beagleortho.co.uk)



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

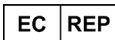

**International Associates Auditing & Certification Limited**

The Black Church, St. Mary's Place, Dublin, D07 P4AX