# ASCEND™ TLSO 456/457 LUMBAR ORTHOSIS

MODEL #SP10456-000

## INSTRUCTIONS FOR USE

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[www.breg.com](http://www.breg.com)

©2019 Breg, Inc. All rights reserved.  
Ascend is a trademark of Breg, Inc.

AW-1.03808 Rev B 05/20

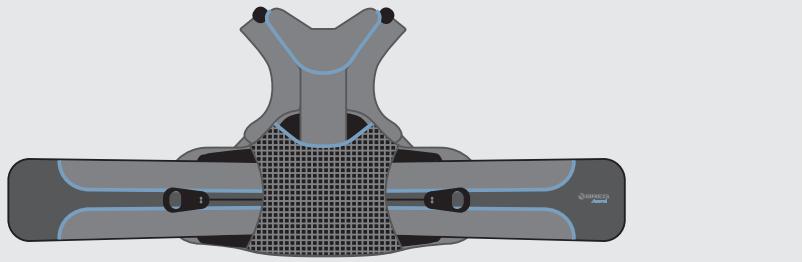
EC REP

E/U authorized representative  
MDSS GmbH, Schiffgraben 41  
D-30175 Hannover, Germany

Size	1	2	3	4	5	6	EXT.
Waist Size	26" – 32" (66.0–81.3cm)	31" – 37" (78.7–94.0cm)	36" – 43" (91.4–109.2cm)	42" – 49" (106.7–124.5cm)	48" – 55" (121.9–139.7cm)	54" – 60" (137.2–152.4cm)	60" + (152.4cm+)
Position	Inner Slot		Outer Slot				

2

**ENGLISH****MEASUREMENT**

- Measure patient's waist and refer to sizing chart ([above](#)) for corresponding size.

**ADJUSTING THE WAIST BELT SIZE**

- NOTE:** The initial configuration is set for size 3, with the belts in the inner slot.
- To adjust the waist belt size, open the brace and lay it face down.
  - Lift one corner of the mesh cover.
  - Flip the pulley plate over the mesh cover.
  - Disengage the belt end from the pulley plate.
  - For sizes 1 to 3, slide the belt panel through the slot and fold the belt over until the end of the belt panel aligns with desired size number. Make sure the belt matches the contour of the pulley plate, and attach the end to the loop.

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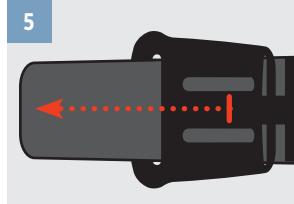
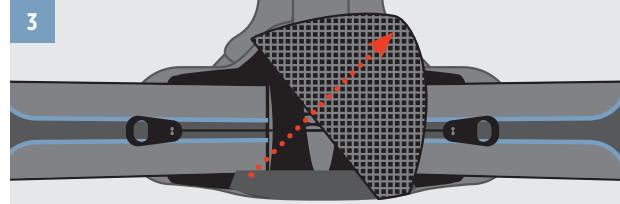
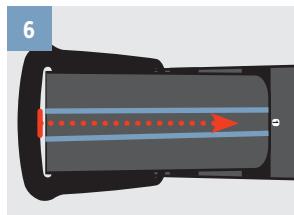
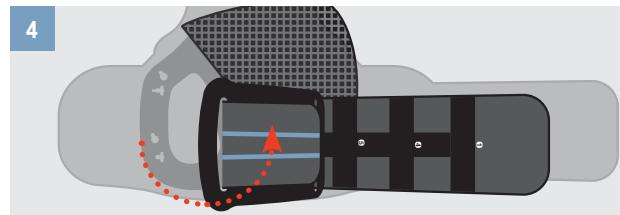
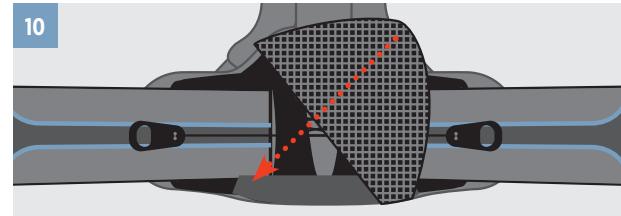
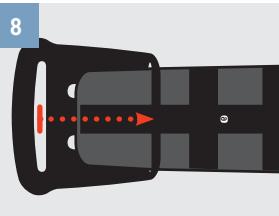
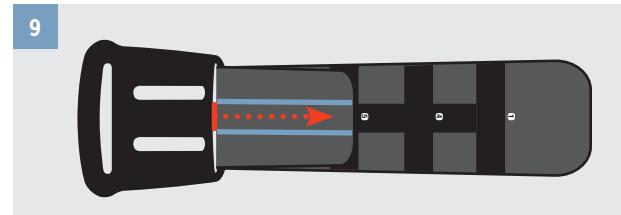
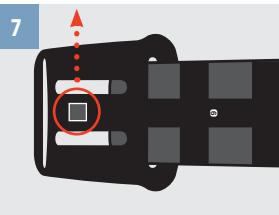
#### ENGLISH

#### **ADJUSTING THE WAIST BELT SIZE (CONTINUED)**

7. For sizes 4 to 6, remove the hook piece attached to the pulley plate.
8. Remove the belt panel from the inner slot and insert it into the outer slot.
9. Slide the belt panel through the slot and fold the belt over until the end of the belt panel aligns with desired size number, and attach the end to the loop on the belt.
10. Flip pulley plate back to starting position. Ensure pulley plate is tucked into bottom flap and top edge of mesh cover. Pull mesh cover tight to attach bottom corner. The inner slot of the pulley plate should be concealed by mesh cover.

11. **Repeat steps 3 through 10** for the other side.

12. Position pull handles on the belts close to the mesh cover. Pull the left and right pulley plates away from each other to center pulley system to the back pad.

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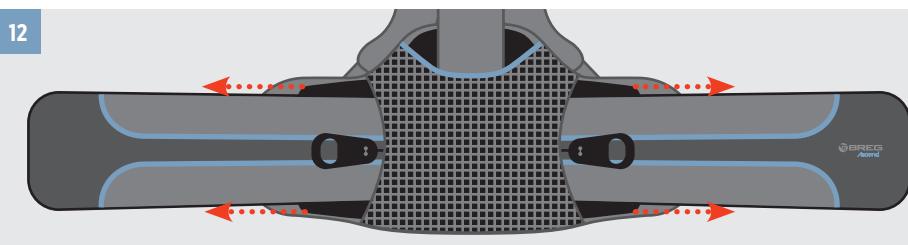
#### 11

**Repeat steps 3 through 10 for the other side**

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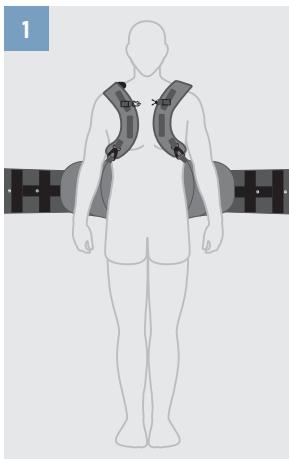
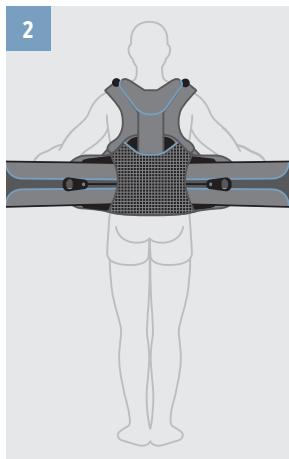
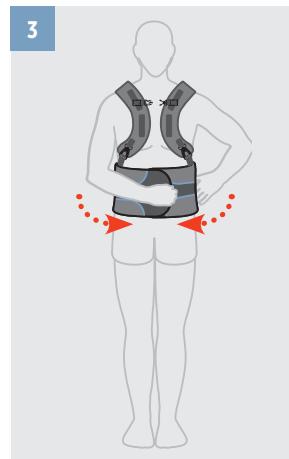
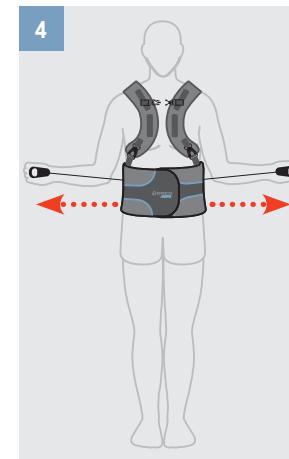
6

**ENGLISH****APPLYING THE BRACE**

Before applying the brace, make sure the pull handles are positioned on the belt, close to the mesh cover on the back of the brace and the sternum strap is unbuckled.

1. Put the brace on by placing the straps over the patient's shoulders.
2. Align the center of the back pad to the patient's spine, and align the bottom edge of the back panel over lumbosacral joint. **Refer to picture 2 for proper orientation.**
3. Fasten the brace around the patient's waist by pulling the right belt over the left belt. Angle belts up or down to accommodate patient's anatomy.
4. Adjust the tension of the brace by pulling the pull handles away from the body.
5. Attach the pull handles to the belt.
6. If no further adjustment is required, buckle the sternum strap and tighten as needed.

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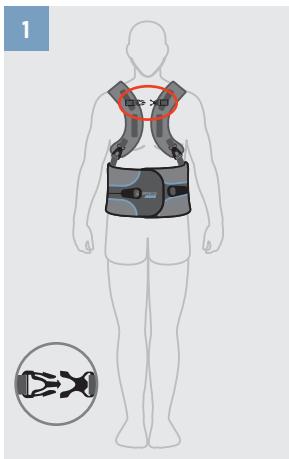
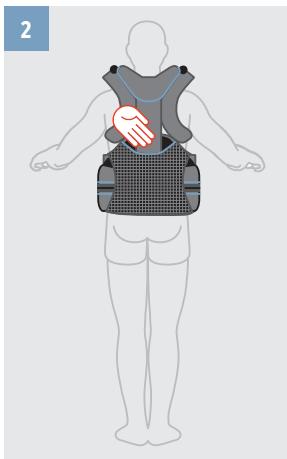
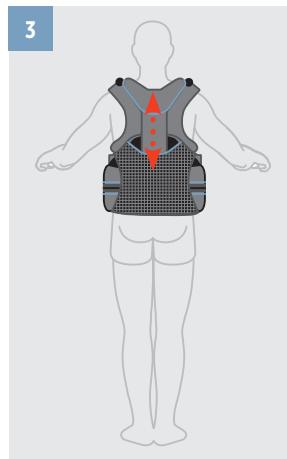
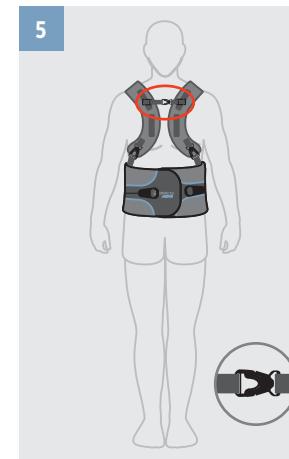
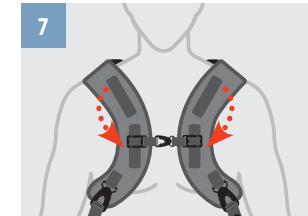
6

7

**ENGLISH****ADJUSTING THE POSTERIOR THORACIC EXTENSION AND STRAPS**  
Make sure the waist belt is properly fitted before adjusting the posterior thoracic extension (PTE).

1. Unbuckle the sternum strap.
2. While the waist belt is on the patient, place hand between the back pad and PTE to separate the hook attachment.
3. Slide PTE to the desired position and secure in place using hook attachment.
4. Ensure the clips at the base of the shoulder straps are buckled. Adjust the straps as necessary to fit the patient.
5. Re-buckle the sternum strap.
6. To adjust the height of the sternum strap, slide the attachment buckles up or down.
7. For further height adjustment of the sternum strap, remove the attachment buckles from the shoulder straps and reapply them onto the lower section on the shoulder straps. Tighten the sternum strap.

**NOTE:** Any excess strap length can be secured using the elastic bands.

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 **WARNINGS** 

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** This device is offered for sale by or on the order of a licensed health care professional.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

**CARE & WASHING INSTRUCTIONS**

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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