

# Hallux Valgus Splint



## INDICATIONS FOR USE

- » Post-operative treatment following Hallux Valgus (Bunion) repair
- » Taylor’s Bunion
- » Hammer toe

## FITTING INSTRUCTIONS

- » Open the hook and loop fastenings on the splint
- » Place the splint under the fore foot with the long steel, starting at the tip of the big toe, lying along the inside of the foot
- » Wrap the splint around the foot and secure the hook and loop fastening
- » Close the fastening around the big toe to hold the toe in a straight line

## WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

### Sizing Information

Size	Male Shoe Size	Femal Shoe Size	Left	Right
Small	-	2 - 4	HVS/3L	HVS/3R
Medium	-	4 - 6	HVS/5L	HVS/5R
Large	5 - 7	6 - 7	HVS/7L	HVS/7R
XLarge	8 - 11	8 - 11	HVS/8L	HVS/8R



### Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR  
E: info@beagleortho.com W: beagleortho.co.uk T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd  
Sutton House, Bishopstown Road  
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.