

Hand / Wrist

Paediatric D-Ring Wrist Brace			
DR/3LR	DR/5LR	DR/7LR	DR/8LR
DR/3RR	DR/5RR	DR/7RR	DR/8RR
DR/3LB	DR/5LB	DR/7LB	DR/8LB
DR/3RB	DR/5RB	DR/7RB	DR/8RB
DR/3LP	DR/5LP	DR/7LP	DR/8LP
DR/3RP	DR/5RP	DR/7RP	DR/8RP
DR/3LG	DR/5LG	DR/7LG	DR/8LG
DR/3RG	DR/5RG	DR/7RG	DR/8RG
DR/3LPNK	DR/5LPNK	DR/7LPNK	DR/8LPNK
DR/3RPNK	DR/5RPNK	DR/7RPNK	DR/8RPNK

Description

An orthosis for support and stabilisation of the wrist. The conformable stay and straps aid stabilisation.

Intended Purpose

To provide wrist support for an array of injuries and conditions, including Pre or Postoperative treatment for: Healing fractures, Strains, Repetitive strain injuries, Juvenile rheumatoid arthritis.

Sizing

Measure the wrist circumference at the narrowest point, choose the correct size for fit and therapeutic benefit.

Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Undo all hook and loop straps
2. Remove the aluminium palmar bar and place the bar along the underside of the wrist onto the palm, bend as required to mimic the anatomical shape
3. Reinsert the palmar bar ensuring it is secured under the loop tab
4. Place the brace on the underside of the wrist and align palmar bar to the palm
5. Fasten hook and loop straps to ensure a snug fit

Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps and remove metal bar
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely
- Before using the brace again, replace the metal bar the correct way round and tuck under the tab of the loop

Product composition

Velfoam, Cotton, Aluminium, Hook and Loop

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.com



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

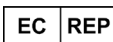

International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX