

Hand / Wrist

Therapy CMC Brace			
THR/303L	THR/305L	THR/307L	THR/308L
THR/303R	THR/305R	THR/307R	THR/308R

Description

An orthosis for support and stabilisation of the wrist and thumb. The conformable stay and straps aid stabilisation.

Intended Purpose

To provide wrist and thumb support for an array of injuries and conditions, including Pre or Postoperative treatment for: CMC ligament sprains/strains/instabilities, CMC support for medial subluxation, Repetitive strain injuries, Rheumatoid arthritis

Sizing

Measure the wrist circumference at the narrowest point, choose the correct size for fit and therapeutic benefit.

Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Undo all hook and loop straps
2. Place the thumb through the thumb sleeve of the support
3. Fasten wrist hook and loop strap around the wrist to ensure a snug fit
4. Place the thumb strap through web space and secure on the hook and loop piece. Trim strap if required.

Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

Product composition

Neoprene, Hook and Loop

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols

 Hand wash

 Dry flat

 Do not tumble dry


 Do not bleach

 Do not dry clean

 Do not iron



Beagle Orthopaedic
Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR
e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.com



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

EC

REP

International Associates Auditing & Certification Limited
The Black Church, St. Mary's Place, Dublin, D07 P4AX