

Upper Limb

Sling And Swathe Support

0653

0655

0657

0658

Description

An orthosis designed to completely immobilise the arm and shoulder. The brace is designed in two parts, the sling and the wraparound padded swathe which secures the arm snugly against the body.

Intended Purpose

To provide upper limb support for an array of injuries and conditions, including Pre or Postoperative treatment for: CVA, Post-trauma, Immobilisation following injury or surgery

Sizing

With the arm at 90° measure the circumference around the arm and body.

▲ Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Place the injured arm into the sling, ensure the elbow and forearm are supported to avoid discomfort. **See Fig.1.**
2. Place the straps at the top of the sling over the shoulders. **See Fig.2.**
3. Cross the straps over at the back and bring the straps around the waist to the front of the sling. **See Fig.3.**
4. Secure both ends of the hook and loop straps to the front of the sling. **See Fig.4.**
5. The straps can be trimmed if required.
6. Position one end of the swathe to the front of the sling securing the velcro. **See Fig.3.**
7. Wrap the swathe all the way around the waist, back and secure the velcro to the sling at the front. **See Fig.**



Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

Product composition

Velfoam, Hook and Loop

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017



International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX