

Spinal

Temporary Hernia Support	
TEMP.09	TEMP.12

TEMP.09	TEMP.12
----------------	----------------

Description

An orthosis designed to offer compression to the abdomen and provide pain relief. The stretch elastic evenly distributes compression to achieve the most comfortable fit and allow for tension control. The easy to position hernia pad can be placed in the optimum place helping to reduce the pain and discomfort

Intended Purpose

To provide abdominal support for an array of injuries and conditions, including Pre or Postoperative treatment for: Hernia's, Umbilical Hernia's, Incisions, Abdominal Contusions, Protrusions, Post-surgery and Abdominal Weakness

Sizing

One size fits most. Available in 2 depth sizes.

Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Place the support in the centre of the back, so that belt sits centrally to the position of the target hernia.
2. Place the hernia pad over the hernia and wrap the support over the pad, bring the side with the hook fastening over the midline and secure down to give equal compression
3. The Temporary Hernia Support should give compressive support to the abdominal area, **the pad can be repositioned as required.**

Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the straps are not over tightened, perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The hook and loop fasteners must always be fastened when the device is not being worn or when being washed
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook and loop fasteners

Washing and care instructions

- Cover all the hook and loop straps and remove metal hinges
- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use
- Periodically check the hook and loop straps are operating securely

Product composition

Velfoam, Elastic, Cotton, Hook and Loop, Latex

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws.

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside.

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron



Contains natural rubber latex




Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com **t:** +44 (0) 1254 268 788 **w:** beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

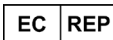

International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX