

Knee & Lower Limb

Patella Knee Strap

PST/BLK

Description


An orthosis designed with non-slip technology, meaning it will not migrate while wearing. Two reinforced stabilisers above and below the patella provide targeted compression while keeping the kneecap in place to prevent displacement without restricting movement and supporting the patella. The two large hook and loop closures provide easy adjustment for the amount of compression needed. The universal design means the strap can be used on either the left or right knee.

Intended Purpose

To provide knee and lower limb support for an array of injuries and conditions, including Pre or Postoperative treatment for: Arthritis, Jumper's knee, Patella Tendonitis.

Sizing

One size fits most, the support is universal in size.

 Fitting Instructions

We recommend the initial fitting of the device be conducted by a suitably qualified Healthcare Professional who will advise fitting, removal and period of use.

1. Place the centre of the brace at the back of the knee
2. Wrap the top strap around the knee and fasten above the patella / knee
3. Wrap the bottom strap around the knee and fasten below the patella / knee
4. Ensure the straps are fastened for a snug fit and not over tightened

Contraindications

- Must not be used by individuals for whom compression is contraindicated
- Do not use on open wounds
- Any known allergy and/or hypersensitivity to any material listed in the product composition
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy

Warnings and Precautions

- Carefully read all instructions and warnings before use
- Follow all instructions to ensure proper performance of the brace
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area
- Do not reuse for another patient, doing so risks cross-infection and can compromise product integrity
- To ensure the brace is not too tight perform regular skin and circulation checks, especially for patients with diabetes, vascular deficiencies and neurological conditions
- Should any adverse reactions occur, please remove, do not use and contact your healthcare professional or provider
- Check brace and all components for wear and tear
- The durability of the device may be compromised by certain factors, e.g. objects with sharp edges

Washing and care instructions

- Hand wash the brace in cool water using a mild detergent
- Rinse and air dry flat
- Do not spin or tumble dry
- Do not use fabric conditioner at any time
- Check for any degradation that may affect use

Product composition

Bioprene, Neoprene, Foam Tubing, Hook and Loop

Storage and transport conditions

Store in a cool, dry place out of direct sunlight, in the original packaging

Recycling and disposal

Packaging and constituent parts should be recycled or safely disposed of in accordance with local or national laws

Serious incident

Report any serious incident to the manufacturer and the competent authority of the EU Member State, or Country in which you reside

Wash symbols



Hand wash



Dry flat



Do not tumble dry



Do not bleach



Do not dry clean



Do not iron




Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

e: info@beagleortho.com t: +44 (0) 1254 268 788 w: beagleortho.co.uk



This Medical Device conforms to the requirements of Regulation (EU) 2017/745 of the European Parliament and of the council of 5 April 2017

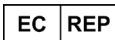

International Associates Auditing & Certification Limited

The Black Church, St. Mary's Place, Dublin, D07 P4AX